 Before use, carefully study the instructions for use. Keep the instructions for use in case of future use.

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Cold and flu are among the most common infectious respiratory tract diseases with significant overall symptoms. They start suddenly, so it's important to intervene quickly and promptly. These diseases (including the common cold) are caused by viruses that are transmitted through infection via mucous membranes into the body, where they begin to rapidly multiply. This is a medical device with an innovative mechanism of action. Thanks to its rich content of polyphenols, the extract from *Cistus creticus* forms a protective film on the nasal mucosa. This protective barrier constitutes a physical (mechanical) defense against viruses and bacteria, preventing their penetration and further multiplication. The site of action of the medical device is on the mucosa of the upper respiratory tract - specifically on the mucosa of the oral cavity for oral spray/pastilles and on the mucosa of the nasal cavity for nasal spray. Other ingredients included - citrus bioflavonoids, purple coneflower (echinacea purpurea), common sage, and Barbados cherry (acerola) strengthen the antiviral neutralizing effect.

Ingredients: *Cistus creticus* extract (1.90 g), purple coneflower (echinacea purpurea) extract (3.06 g), common sage (*Salvia officinalis*) extract (0.88 g), citrus bioflavonoid complex (0.05 g), acerola extract (0.02 g), hyaluronic acid, mint flavor, Barbados aloë extract, water, phenoxethanol, ethylhexylglycerin

Intended use: Fytofontana VIROSTOP is intended as a protective barrier against viruses causing inflammation of the upper respiratory tract by application to the mucosa of the upper respiratory tract.

Indications: Redness of the mucosa in upper respiratory tract inflammation.

Main mechanism of action: The main mechanism of action of the medical device fytofontana VIROSTOP nasal spray lies in its ability to cover the mucous membranes of the nasal cavity with a protective film, thereby preventing the penetration of viruses into the body.

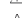
Contraindications: Allergy to any of the ingredients of the medical device (listed in the Ingredients section).

Side effects: When using fytofontana VIROSTOP nasal spray, you may experience temporary mild irritation of the mucosa. These sensations are temporary and natural.

Intended users/patients: The medical device fytofontana VIROSTOP nasal spray is intended for adults and children over 6 years old.

Limitations of use for the population: The duration of use of the medical device depends on the persistence of symptoms but should not exceed 7 days.

Preventive measures and warnings:

-  Do not use in children under 6 years of age. Older children should use the product only under the supervision of an adult.
-  For pregnant and breastfeeding women, consultation with a medical specialist regarding usage is recommended.
-  We do not recommend using the product if you have had surgical procedures in the nose of the mouth area, or if the mucous membrane of the upper respiratory tract has been damaged.
-  The product is intended for single-person use to reduce the risk of infection transmission.
-  Keep out of reach of small children.

Any slight sedimentation is caused by the plant-based ingredients contained in the product and does not affect the quality of the product in any way. When handling the spray, prevent direct contact with the eyes. If the product comes into contact with the eyes, rinse with running water.

Dosage:

Preventive: Apply 3-4 x daily, 1 spray into each nostril. Children older than 6 years receive 2 x daily, 1 spray into each nostril. It is suitable for long-term preventive use. In case of higher risk of infection, the spray can be applied multiple times daily.

In case of acute condition: Apply 4-5 x daily, 1 spray into each nostril. For children older than 6 years, administer 3 x daily, 1 spray into each nostril.

It is very important to apply in the evening before bedtime.

The main benefit for the patient lies in the impact on public health through the use of VIROSTOP, where:

- Polyphenols support the local immunity of the upper respiratory tract in the early stage of infection.
- Improvement in quality of life by reducing the duration of symptoms causing upper respiratory tract infections.
- Preventive use of VIROSTOP during periods of increased infection prevalence in communities reduces susceptibility to risks of upper respiratory tract infections.

Method of using the spray:

- The spray form allows for easy and even application to the nasal mucosa.
- Shake well before use.
- Remove the protective cover from the applicator.
- Insert the applicator into the nostril and administer by pressing the spray head.
- After each use, clean the nasal applicator and replace the protective cover.

Package contents: Bottle with a volume of 20ml, spray with applicator

Lifespan of the medical device: The lifespan of the medical device is determined by the expiration date on the product packaging - 36 months from the date of manufacture.

Accessory or compatible medical devices: Fytofontana VIROSTOP medical device does not have any accessories or compatible medical devices.

Storage:

-  The information is provided on the packaging

Disposal: Return any unused product to the pharmacy. Dispose of clean packaging materials in the appropriate recycling bin.

 Herb-Pharma Corporation s.r.o., 935 65 Veľké Ludince, Slovak Republic, under license from Herb-pharma AG Schweiz, Waldmannstrasse 6, 8001 Zürich, Switzerland

www.virostop.info
www.fytofontana.com

Residual risks, limitations of use, further information. Any suspicions of adverse events or adverse events caused by the device should be reported by the user to the manufacturer and the relevant authority in the member state where the user resides. The manufacturer conducts a detailed risk analysis of the evaluated medical device as a continuous process during design and development and in the production phase of the medical device according to EN ISO 13485:2016 and Regulation 2017/745 MDR. The risk analysis according to EN ISO 14971:2019 has been developed for the evaluated medical device. The risk analysis is regularly updated based on the results of post-market surveillance of the medical device /PMS/.

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